**medi GmbH & Co. KG**  
 Medicusstraße 1  
 D-95448 Bayreuth  
 Germany  
 T +49 921 912-0  
 F +49 921 912-780  
 ortho@medi.de  
 www.medi.de

medi Australia Pty Ltd  
 83 Fennell Street  
 North Parramatta NSW 2151  
 Australia  
 T +61-2 9890 8696  
 F +61-2 9890 8439  
 sales@mediaustralia.com.au  
 www.mediaustralia.com.au

medi Austria GmbH  
 Adamgasse 16/7  
 6020 Innsbruck  
 Austria  
 T +43 512 57 95 15  
 F +43 512 57 95 15 45  
 vertrieb@medi-austria.at  
 www.medi-austria.at

medi Belgium bvba  
 Posthoornstraat 13/1  
 3582 Koersel  
 Belgium  
 T: + 32-11 24 25 60  
 F: +32-11 24 25 64  
 info@medibelgium.be  
 www.medibelgium.be

medi Brasil  
 Rua Cristovan de Vita, n.º 260, Un. 30,  
 Centro Logístico Raposo Tavares,  
 Vila Camargo  
 Vargem Grande Paulista - SP  
 CEP 06730-000  
 Brasil  
 T: +55-11-3500 8005  
 sac@medibrasil.com  
 www.medibrasil.com

MAXIS a.s.,  
 medi group company  
 Slezská 2127/13  
 120 00 Prague 2  
 Czech Republic  
 T: +420 571 633 510  
 F: +420 571 616 271  
 info@maxis-medica.com  
 www.maxis-medica.com

medi Danmark ApS  
 Vejlegardsvej 59  
 2665 Vallensbaek Strand  
 Denmark  
 T +45-70 25 56 10  
 F +45-70 25 56 20  
 kundeservice@sw.dk  
 www.medit danmark.dk

medi Bayreuth Espana SL  
 C/Canigo 2-6 bajos  
 Hospitalet de Llobregat  
 08901 Barcelona  
 Spain  
 T +34-932 60 04 00  
 F +34-932 60 23 14  
 medi@mediespana.com  
 www.mediespana.com

medi France  
 Z.I. Charles de Gaulle  
 25, rue Henri Farman  
 93297 Tremblay en France Cedex  
 France  
 T +33-1 48 61 76 10  
 F +33-1 49 63 33 05  
 infos@medi-france.com  
 www.medi-france.com

medi Hungary Kft.  
 Bokor u. 21.  
 1037 Budapest  
 Hungary  
 T +36 1 371-0090  
 F +36 1 371-0091  
 info@medi.hu  
 www.medi.hu

medi Nederland BV  
 Heusing 5  
 4817 ZB Breda  
 The Netherlands  
 T +31-76 57 22 555  
 F +31-76 57 22 565  
 info@medi.nl  
 www.medi.nl

## Epicomed®

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### Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the product, please consult your doctor or orthotist immediately. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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medi Trading (Shanghai) Co. Ltd.  
Rm 1806, Building 1 Sandhill Plaza  
No. 2290 Zuchongzhi Road  
Pudong New District  
200131 Shanghai  
People's Republic of China  
T: +86-21 61761988  
F: +86-21 61769607  
info@medi.cn

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS  
Kültür mah. çaldıran sok.14/3  
06420 Çankaya/Ankara  
Turkey  
T +90 312 435 20 26  
F +90 312 434 22 82  
info@medi-turk.com  
www.medi-turk.com

medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL  
Great Britain  
T +44-1432 37 35 00  
F +44-1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Ukraine LLC  
Tankova Str. 8, office 35  
Business-center „Flora Park“  
Kiev 04112  
Ukraine  
T +380 44 591 11 63  
F +380 44 392 73 73  
info@medi.ua  
http://medi.ua

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000  
USA  
T +1-336 4 49 44 40  
F +1-888 5 70 45 54  
info@mediusa.com  
www.mediusa.com

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Polyamid, Polyester, Elasthan, Viskose

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# Epicomed®

## Intended purpose

Epicomed is an elbow support. The product is exclusively to be used for the orthotic fitting of the elbow and only on unbroken skin or if any wounds have been properly covered.

## Areas of use

- Irritation at the tendon insertion (Epicondylitis humeri), e.g. tennis elbow
- Joint inflammation (arthritis)
- Painful joint wear (arthrosis)
- Irritation of the joint (e.g. excessive stress, injuries or operations) linked to swelling or bruising

## Side-Effects/Contraindications

No side-effect is known which influences the whole body. However, if bandages (or similar aids) are too tight, local pressure effects or constriction of blood vessels or nerves may occur in individuals. For this reason, you should discuss this with your doctor before applying the bandage if you suffer from:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as unclear swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression arm stockings) must be discussed in advance with the responsible doctor.

## Mode of Action/Comment on Wear

Epicomed causes a massage effect, which supports perfusion, by interacting with compressive knitted fabric and the two integrated silicon inlays. This effect furthers the decline in swellings and bruises. Irritation and pain are relieved. In addition, the lashing strap can be individually adjusted for the level of stress at work or during sport. This helps to reduce excessive stress on the musculature and tendons.

Supports cause the greatest effect during physical exercise. In principal, the support can be worn all day. However, this depends on how comfortable the support is to wear. You should avoid bending your arm tight for a long period. The support should be taken off for longer breaks (e.g. when sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialized shop.

The support is suitable for use in water. After use, please rinse out the product thoroughly.

## Putting on/Taking off

Please note that initial fitting should be supervised by a qualified healthcare professional.

- To make putting Epicomed on easier, fold down the upper fastening edge first. (Fig. 1)
- Pull the bandage over the elbow so that the crease of the elbow sits in the middle of the knitted comfort zone and the seam on the back side runs over the middle of the elbow bone

at the back of the arm. Then roll the upper part of the bandage back up. (Figs. 2 + 3)

- The position of the bandage is correct when the inner and outer bone (epicondyles) can be clearly felt in the gaps in the silicone inserts. (Figs. 4 + 5)
- The tension strap enables the pressure to be regulated for each individual, depending on what their body requires (tightening, loosening). If required, the strap can also easily be removed whilst the device is being worn. (Fig. 6)

---

### Care instructions

Velcro fasteners should be closed for washing. We recommend using a laundry bag. Fabric softeners, fats, oils, lotions, ointments and soap residues can affect the material, hinder the Clima Comfort effect and cause skin irritations.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

---

### Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

### Material composition

Polyamide, polyester, elastane, viskose

### Liability

The manufacturer's liability will become void if the product is not used

as intended. Please also refer to the corresponding safety information and instructions in this manual.

### Disposal

The product can be disposed of in the domestic waste.

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